

Universal Post Remover



Universal Post Remover, a modified version of the Gonon Post Remover, was designed to address even the most challenging post removal situations. Stainless steel provides improved performance. It can be used on posts made of steel, gold, stainless steel, or fiber.

The color-coded system allows you to remove posts in three easy steps: (1) To center the trephine, a special diamond bur is included to taper the protruding head of the post. (2) The color-coded trephine is used to bore and gauge the protruding post to the exact size of a corresponding tap of the same color code. The tap is screwed manually, counter-clockwise (screw post will be unscrewed). (3) The extracting plier is fixed on the tap and the jaws of the pliers are expanded by tightening the knurled knob. This procedure will separate the post from the tooth quickly and safely.

The kit contains 1 FG Diamond Bur, 1 Pointeau drill, 2 sets of 4 trephine bur sizes from 1.15mm to 1.60mm inside diameter, 2 sets of 4 tubular CCW taps, a set of washers, and pliers with screwing knob.

*Dent Corp. Research & Development
7-11 S. Broadway
White Plains, NY 10601
Phone 800-454-9244; 914-682-6600
E-mail dentcorp@cloud9.net*